

KETOSCAN^{MINI}

Real-time fat burning monitor



SCAN YOUR KETOSIS LEVEL

By breath sampling



GET IT ON
 Google Play

 Download on the
App Store

It's far more than a exhaled breath —

■ Ketoscan Mini

measures a concentration of acetone in your exhaled breath when your body fat is burning. So it provides you how much of true body fat is burning now and enables you to establish an affordable food and exercise plan.



Non-invasive

Do not hurt yourself anymore to see a blood. Just breath out smoothly.



Handy & Portable

Bring this pocket-sized device wherever you go.



Hygienic use

Replaceable disposable mouthpiece whenever you testing.



Mobile app. connectable

Track and monitor your data history and make a plan of exercise intensity

■ Ketoscan mini will be useful for

- LCHF / Ketogenic / Paleo diet
- Low calorie diet
- Any type of weight loss exercise or program
- Medical treatment by Ketogenic food therapy



■ The figures of your ketosis state are shown in a display



■ The meaning of indications

■ 2.0 ~ 3.9

Average 2.0g/hour of body fat is burning

■ 4.0 ~ 9.9

Average 5.0g/hour of body fat is burning

■ 10 ~ 39

Average 10.0g/hour of body fat is burning
Do not maintain this level in a long-term

■ 40 ~ 60

Body fat is burning too fast
It could be Ketoacidosis state
Should go see a doctor

■ 60 ~

It is Ketoacidosis state
and serious risks to your health



Product Specification

Sensor type

High resolution acetone gas sensor

Indication of result

PPM (Parts per million)

Display resolution

0.1 PPM until 10PPM /
1 PPM above 10PPM

Display range

0.0 ~99 PPM

Accuracy level

10%

Warming up time

Max. 60 seconds

Analyzing time

In 5 seconds

Power

Rechargeable
lithium-polymer battery

Testing numbers with full charge

Min. 50 tests

Calibration interval

300 tests

Mouthpiece

Disposable mouthpiece

Size

2.12x4.33x0.72 inches /54x110x18.5mm

Weight

0.16lbs / 74g


Storage / Working temperature

41~122°F (5~50°C) / 50~104°F (10~40°C)

Certification

FDA Class I

Bluetooth® Pairing with mobile App. KETOSCAN

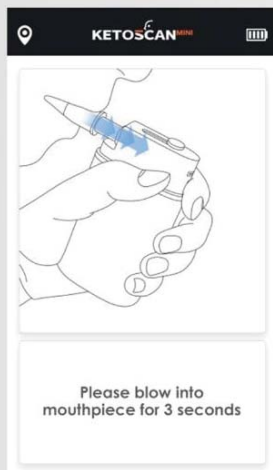

Instructions how to use

Simple and easy to follow.



Your breath acetone analysis

Check out the speed of body fat burning and blood ketone level based on your breath acetone level.

And App. will give you a comment about your body fat burning state.



Track and monitor

Which factors affects to your today's low/high breath acetone level?

Make a meal & exercise plan for the sustainable ketosis state.

Head office

SENTECH GMI Corp.
1854 18th floors 382 Gangnam-daero
Gangnam-gu Seoul SOUTH KOREA 06232
Tel: +82 2 6929 1310

Germany

Ludwig-Erhard-Str.30~34,
D-65760, Eschborn, GERMANY
Tel : +49(0)6196 761 0600
Fax : +49(0)6196 761 1763

USA

SENTECH USA INC
20280 S.Vermont Ave. STE 200
Torrance, CA 90502
Tel : +1 310 526 8231
www.sentech-usa.com
sales@sentech-usa.com