KETOSCAN^{MINI}

Real-time fat burning monitor

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Indstilles under settings i appen.

Q7 I want to change the language in the app. Should I download another app?

KETOSCAN MINI dedicated app shows other language according to the configuration of the smart phone. Please reboot the app after change the smartphone configuration as the language want to change.

SENTECH GMI Corp.

722 7th floors 161 Yanghwa-ro

Mapo-gu Seoul SOUTH KOREA 03995

Tel: +82 2 6929 1310

www.ketoscan.com

sales@sentechgmi.com

Warranty

FCC compliance information

THIS DEVICE COMPLIES WITH PART 15 OF THE FCC RULES.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Modifications not expressly approved by the manufacturer may void the user's authority to operate the equipment under FCC rules.

Limited ONE year warranty

Warranty coverage

SENTECH GMI Corp. warrants this product against defects in materials and workmanship under normal use for a period of ONE (1) YEAR from the date of retail purchase by the original end-user purchaser ("Warranty Period"). Except as provided in the preceding sentence or applicable law, SENTECH GMI Corp. provides this product "as is". If a defect arises and a valid claim is received by SENTECH GMI Corp. within the Warranty Period, at its option, SENTECH GMI Corp. will either (1) repair the product, (2) exchange the product with a product that is new or which has been manufactured from new or serviceable used parts and is at least functionally equivalent to the original product, or (3) refund the purchase price of the product. When a refund is given, the product for which the refund is provided must be returned to SENTECH GMI Corp. or Seller and becomes SENTECH GMI Corp. or Seller property. This warranty is limited to the original end-user purchaser and is not transferable to, or enforceable by, any subsequent owner.